


Circadiance® SleepWeaver® Fitting Guide

Facial Measurements for Nasal Masks (ruler needed – see fig. 1)

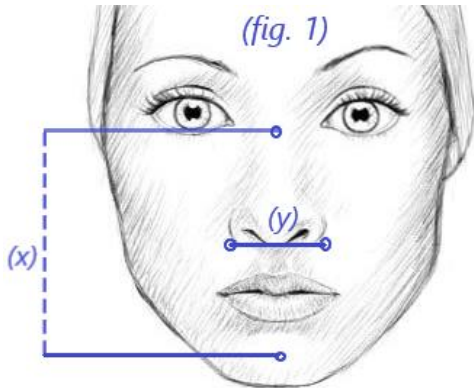

To determine your SleepWeaver nasal mask size, measure the width at the bottom of your nose, edge to edge. Enter the width (y) in inches (round to closest quarter): _____ in.

NOTE: It may be helpful to have a friend assist you. For accurate results of the face, do not use a flexible measuring device such as a sewing measuring tape or a string.



Elan—*The Elan comes in three sizes to fit most faces.*

If the measurement under your nose is 1 inch or below, try the **Small Elan**.

If the measurement under your nose is 1 to 1½ inches, try the **Regular Elan**.

If the measurement under your nose is 1½ inches or more, try the **Large Elan**.



Advance & Advance Small—*This mask is designed to be “one size fits most”.*

If the measurement underneath your nostrils is 1¼ inches or more, try the **Regular Advance**.

For more petite faces, if you measure 1 inch under your nares, try the **Advance Small**.



Advance Pediatric—*This mask is designed to fit children ages 2–7 years old—no sizing necessary!*



3D—*The cushion is one size fits most.*

For neck measurements 16 inches and below—if you don’t know your size, use a *flexible* measuring device like a sewing measuring tape or string—try the **Regular 3D**.

If your neck is 16½ inches or more, try the **Large 3D**.