

## QUICK START GUIDE

### ▲ IMPORTANT!

- When using the Pilot-12 Lite or Pilot-24 Lite as a portable PAP battery, please disconnect the heated humidifier and heated hose from your PAP device as this will significantly decrease the battery run time!
- Fully charge the battery before storage and a minimum every 6 months while stored!
- DO NOT expose the battery to temperatures above 40°C / 104°F or below -10°C / 14°F. Exposing the battery to very low temperatures can cause damage to the lithium ion cells and exposing the battery to very high temperatures can be a fire hazard!


For technical support and questions about this product please email to [support@medistrom.com](mailto:support@medistrom.com) or contact us by phone:

North America: 1-888-407-4791

Australia: 1300-146-358

Europe: +44 800 060 8576

### PILOT LITE CONTROL PANEL

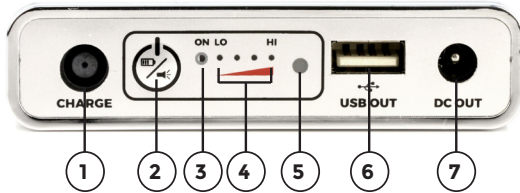

1. Charge Port
2. Multi Function Button
3. Blue "ON" Indicator Light
4. Green Charge Level Indicator Lights
5. LED Flashlight
6. USB Port (5V 2.5A Out)
7. DC Out Port

### SETTING UP YOUR PILOT-12 LITE

#### STEP 1



Plug the original Philips DreamStation or Phillips System One S60 power adapter into the Charge Port of the Pilot-12 Lite and plug the other end into the mains power outlet. (Please fully charge your Pilot-12 Lite before using for the first time).

**NOTE:** If you are using the following compatible 12V PAP device AC Power Adapters, please use the appropriate Short Charging Adapter cables:

- DeVilbiss IntelliPAP I (not included, SKU P12MXDPLK)
- HDM Z1 (not included, SKU P12MXDPLK)



#### STEP 2



Plug the L-shaped yellow tip of the corresponding DC Output cable for your PAP device into the DC Out port of the Pilot-12 Lite. Then connect the other end into your compatible 12V PAP device.



For help with disabling or disconnecting your heated humidifier and heated tubing please visit [www.medistrom.com/support.html](http://www.medistrom.com/support.html)

### SETTING UP YOUR PILOT-24 LITE

#### STEP 1



Plug the original ResMed AirSense 10 AC power adapter into the Charge Port of the Pilot-24 Lite and plug the other end into the mains power outlet. (Please fully charge your Pilot-24 Lite before using for the first time).



**NOTE:** If you are using the following compatible 24V PAP device AC Power Adapters, please use the appropriate Short Charging Adapter cables:

- ResMed S9 series (not included, SKU P24R09PLK)
- Apex Medical iCH series (not included, SKU P24AMIPLK)

#### STEP 2



Plug the L-shaped yellow tip of the corresponding DC Output cable for your PAP device into the DC Out port of the Pilot-24 Lite. Then connect the other end into your compatible 24V PAP device.



For help with disabling or disconnecting your heated humidifier and heated tubing please visit [www.medistrom.com/support.html](http://www.medistrom.com/support.html)